


EVAC+CHAIR GUIDE FOR ASSISTED STAIRWAY EVACUATION


Providing disaster readiness equity and access

1

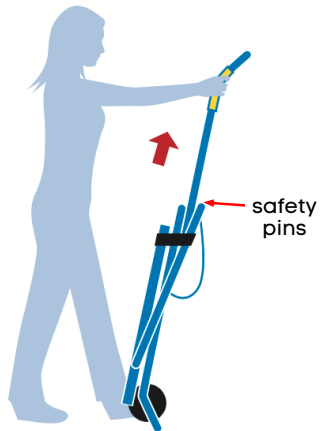

Raise the extension handle until the safety pins lock.

2

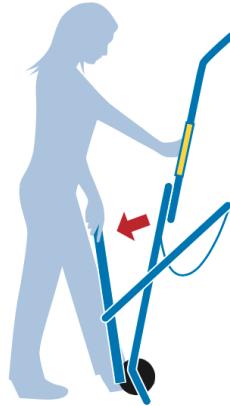

Unbuckle the safety belts, and extend the back leg.

3



Pull the rear safety wheels into place with foot.

4




Once chair is secured with four wheels on ground, assist person into chair and fasten safety belts.

5



Tilt chair forward and push in rear wheels with foot before beginning descent.

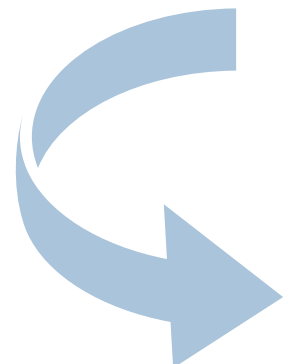

Turn page over for Steps 6 to 9.

EVAC+CHAIR GUIDE FOR ASSISTED STAIRWAY EVACUATION


Providing disaster readiness equity and access

6



Tilt chair backwards until the wheels engage. Push the chair onto the stairs and lower to match the angle of stairs.

7

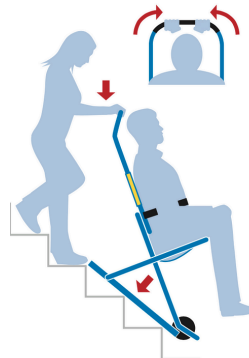


Guide chair down stairs. Weight will be transferred onto the stairs with minimal required effort from person guiding it.

8



Once at the bottom of the stair, tilt the chair onto the bar and pull the safety wheels out with foot.

9

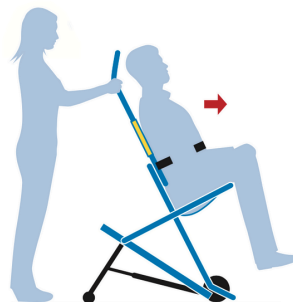

Assist person from the chair to their personal mobility device.

The Evac+Chair can be re-deployed if more people need assisted stairway evacuation.